KoolSling™

FITTING INSTRUCTIONS

ANLEITUNG ZUM ANLEGEN
ISTRUZIONI PER
L'ADATTAMENTO
MISE EN PLACE

E/U authorized representative
MDS
Burckhardtstrasse 1
D-30163 Hanover
Germany


BREG Inc.
Vista, CA 92081 U.S.A.

Telephone (800) 321-0607
(760) 599-3000

Telefax (760) 598-6193

PN 1.00136 Rev A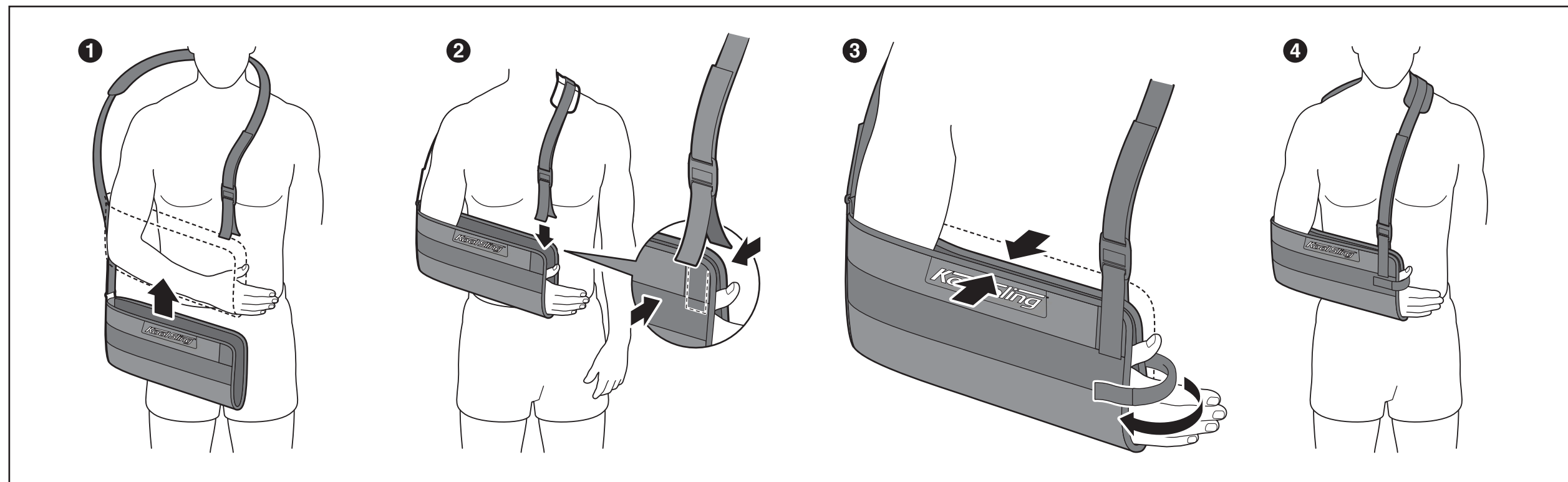

⚠ WARNINGS ⚠

CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.

CAUTION: FOR SINGLE PATIENT USE ONLY.

CAUTION: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

WARNING: CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE.

WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.

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FITTING INSTRUCTIONS

- 1 Slip the Sling onto the forearm with the patient's elbow tucked snugly into the bottom, closed end of the Sling. The Sling is fully universal (left or right arm). Place the adjustable strap around the patient's neck and shoulders. Position the adjustable pad behind the neck.
- 2 Attach the strap end to the Sling using the 'alligator strap tab' to close and suspend the Sling at the open corner near the hand.
- 3 Close the top edge of the Sling using the hook and loop tabs inside the top edge of the Sling. Attach the thumb strap to the Sling; position the strap between the thumb and fingers.
- 4 Properly fitted Sling.

CARE AND CLEANING

Straps and Sling must be removed and cleaned regularly depending upon your activity level and perspiration. Hand wash straps and Sling in cold water with mild detergent, rinse, AIR DRY.

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