SlingShot2™

FITTING INSTRUCTIONS

ANLEITUNG ZUM ANLEGEN
ISTRUZIONI PER L'ADATTAMENTO
MISE EN PLACE
INSTRUCCIONES DE AJUSTE


BREG Inc.
Viso, CA 92081 U.S.A.

Telephone (800) 321-0607
(760) 599-3000
Telefax (760) 598-6193

PN-1.16006 REV A

E/U authorized representative
MBSS
Burckhardtstrasse 1
D-30163 Hannover
Germany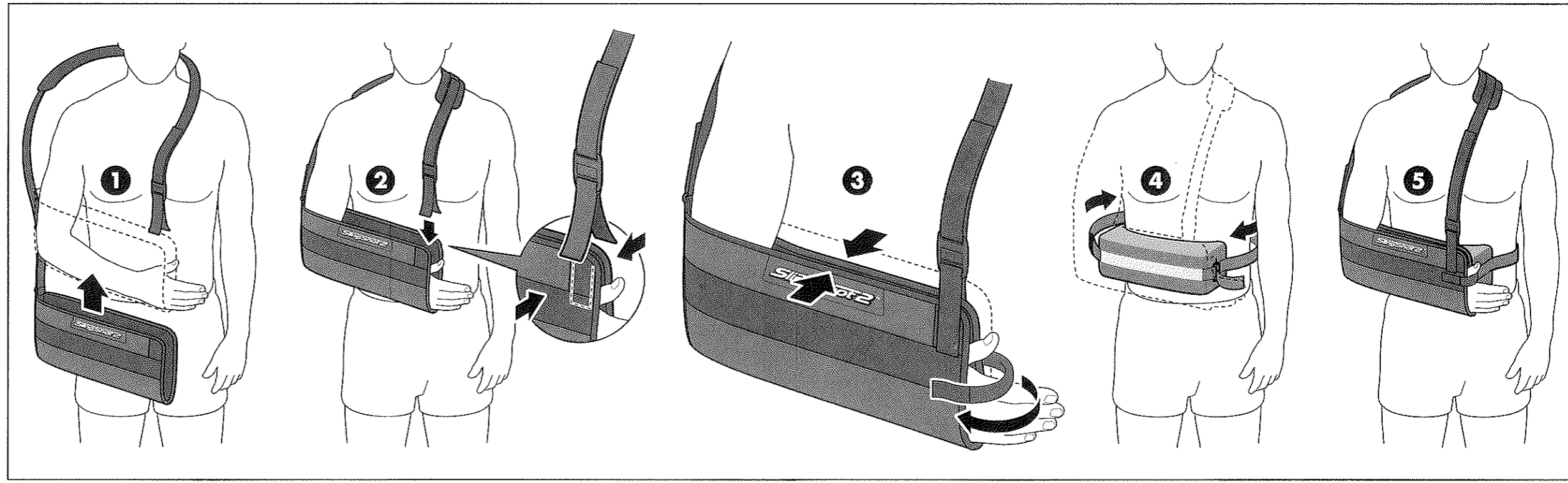

⚠ WARNINGS ⚠

CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.

CAUTION: FOR SINGLE PATIENT USE ONLY.

CAUTION: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

WARNING: CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE.

WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.

FITTING INSTRUCTIONS

- Slip the Sling onto the forearm with the patient's elbow tucked snugly into the bottom, closed end of the Sling. The Sling is fully universal (left or right arm). Place the adjustable strap around the patient's neck and shoulders. Position the adjustable pad behind the neck.
 - Attach the strap end to the Sling using the 'alligator strap tab' to close and suspend the Sling at the open corner near the hand.
 - Close the top edge of the Sling using the hook and loop tabs inside the top edge of the Sling. Attach the thumb strap to the Sling; position the strap between the thumb and fingers.
 - Place the pillow at the waistline. Attach the strap and adjust to fit. Attach the Sling to the pillow, lining up the hook and loop on the pillow with the sling.
- NOTE: The pillow is designed to keep the injured arm in 15° of abduction. The position of the pillow can be adjusted for internal or external rotation by sliding/rotating the pillow forward or backward along the waistline.
- Properly fitted Sling and pillow.

CARE AND CLEANING

Straps and Sling must be removed and cleaned regularly depending upon your activity level and perspiration. Hand wash straps and Sling in cold water with mild detergent, rinse, AIR DRY.

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